

# **SHOPRIDER®**

**ATTENTION**

Please read your  
owner's manual before  
driving your scooter.



**ROCKY 12**

**Owner's Manual**



## CONTENTS

INTRODUCTION .....	3
FEATURE GUIDE .....	3
SAFETY ADVICE .....	4
EMI WARNING .....	6
ADJUSTMENTS .....	8
SCOOTER TILLER CONTROLS .....	9
FREE WHEEL OPERATION .....	12
STARTING TO DRIVE YOUR SCOOTER.....	13
EXTRA GUIDELINES IN THE CONTROL OF YOUR SCOOTER .....	14
BATTERIES AND BATTERY CHARGER .....	15
MAINTENANCE .....	17
TROUBLE SHOOTING GUIDE .....	19
WARRANTY TERMS & CONDITIONS .....	20
SPECIFICATIONS.....	22
NOTES .....	23
DISCLAIMER .....	24

# SHOPRIDER®


## INTRODUCTION

Congratulations on your purchase of the Shoprider® scooter. This scooter incorporates the latest innovative designs patented and only available on a Shoprider® scooter. The large tires, powerful motor and full suspensions ensure a comfortable ride. This Shoprider® scooter is designed for persons with limited mobility.

***Please read this manual carefully and thoroughly before using your scooter and make sure you understand all the instructions.***

## FEATURE GUIDE (ROCKY 12)

1. Tiller Handle Bar
2. Rear Mirror
3. Headlight /Taillight /Indicator Light / Horn Lever
4. Turn Signal
5. Headlight
6. Black Pneumatic Tire
7. Footrest Mat
8. Seat Reclining
9. Armrest
10. Vinyl Captain Seat
11. Taillight
12. Reflectors
13. Anti-tip Wheels
14. Charging Port
15. Storage Pocket
16. Handbrake

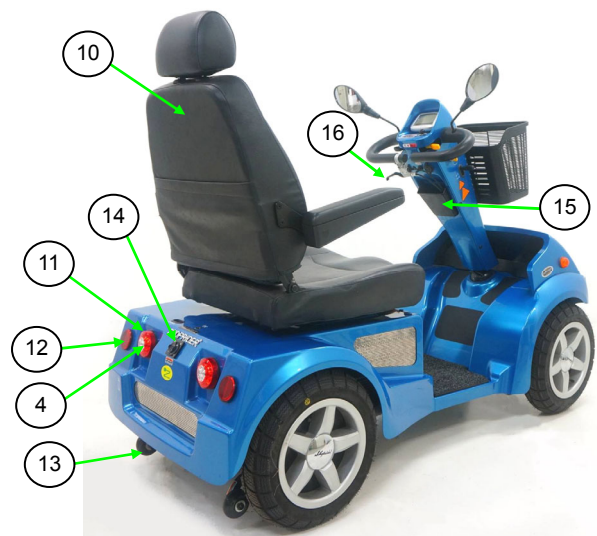



## **SAFETY ADVICE**

- Do not carry passengers.
- This mobility Product is intended to assist you. For your own safety, please read the Owner's Manual before using the Scooter. If in any doubt, contact your supplier directly for advice.
- Driver weight may exceed weight of the unit; speed must be reduced when turning.
- Do not get on or off the scooter unless the brake is on and the scooter is switched off.
- Do not switch off the scooter while the scooter is moving.
- Do not drive the scooter where you can not safely or legally walk.
- Do not attempt to drive over curbs greater than 60mm in height. Doing so could cause your scooter to turn over, causing personal injury or damage to the scooter. For curbs less than 60mm, always approach them at an angle of 90 degrees and at a low speed.
- Do not drink or use a mobile phone when driving your scooter.
- Do not exceed the suggested user weight limit to use the product. Doing so, could cause personal injury and/or damage to the unit or surrounding property may occur.
- Always keep your feet on the scooter when moving.
- Please make sure that you are physically fit to operate this unit. Consult your dealer/doctor for advice.
- Always slow down and proceed carefully when traveling over rough or uneven ground.
- For individuals with balance problems, practice getting on the scooter with the assistance of a qualified healthcare professional.
- Do not operate the scooter without anti-tip wheels installed.
- Do not use an escalator to move a scooter between floors or serious injury could occur.
- Do not drive the scooter on an incline that has oil, water or ice film on it.
- Do not try to lift the scooter by its seat, tiller, rear chassis cover, or any removable parts. Any of these could cause injury to the user or damage to the scooter.
- Do not use parts or accessories that are not authorised by the original manufacturer of the scooter.
- Before sitting in or exiting the scooter, REMOVE the key from the ignition. This will insure the POWER is off and the scooter can not move under its own power. As a general precaution, always remove the key from the ignition while the scooter is not in use. This will prevent the use by untrained and unauthorised persons.
- Do not connect any foreign device to the scooter battery or charging port. This could cause failure of the device, the scooter and personal injury.



- Make sure the tiller is properly adjusted and in the locked position to suit the user to avoid damage and injury.
- It is not allowed to have any person sitting in the scooter when transported by a vehicle, otherwise any injury or damage that might result.
- Do not use the scooter in the rain, near a shower or in a damp bathroom. Direct exposure to rain or dampness could cause the scooter to electrically malfunction.
- Do not lean against or pull forward on the tiller while mounting or dismounting the scooter. Otherwise, injury may occur.
- Do not operate the unit or turn the key on while charging.
- Whenever a center bolt of the wheel assembly has been loosened, please replace a specified new bolt from authorized dealers and secure with a torque of  $240 \pm 5$  kg-cm together with Loctite 271 adhesive (or equivalent).
- Charge batteries daily after use.
- Charging plug has to be removed from the scooter after charging.
- Battery cables must be securely fastened onto battery terminals by the authorised dealers before using the unit and secured inside the scooter rear cover.
- Batteries must be tied down correctly and securely with the straps supplied.
- Any loose cable can not be allowed when the unit is driving.
- Always check the battery charge remaining before operating the unit to ensure you have sufficient for your journey.
- Always charge the batteries of the unit in a well-ventilated area.
- If the unit went over or sank into deep water, please switch off power of the unit and consult with your authorized dealer for help right away.
- Switch off power of the unit while not in use or before charging, removing and installing the batteries. Otherwise, injury and/or damage to the unit or surrounding property may occur.
- Do not operate the unit in the way of exceeding the product's specification listed in this owner's manual. Otherwise, injury and/or damage to the unit or surrounding property may occur. Doing so may also Void your warranty.
- Do not use the thumb/finger lever before the key switch is turned on, or the operation will be prohibited until to turn key switch off and then on.
- The speed dial must adjust under 6 to lower the speed when driving down the slope to avoid the injury of the occupant or damage on any components on the scooter.
- Ensure the scooter is serviced every 12 months by an authorised Shoprider service provider to ensure it remains in safe working order. Failure to do so may cause catastrophic failure with possible injury and damage to surrounding property



## EMI Warning

Shoprider® scooters may be susceptible to electromagnetic interference (EMI), which is a kind of interfering electromagnetic energy (EM) emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitters, two-way radio, and cellular phones. The interference (from radio wave sources) can cause the scooter to release its brakes, move by itself, or move in unintended directions. It can also permanently damage the scooter's control system. The sources of radiated EMI can be broadly classified into three types:

1. Hand-held portable transceivers (transmitters-receivers) with the antenna mounted directly on the transmitting unit. Examples include: citizens band (CB) radios, "walkie talkie", security, fire and police transceivers, cellular telephones, and other personal communication devices.

**NOTE! Some cellular telephones and similar devices transmit signals while they are ON, even when not being used.**

2. Medium-range mobile transceivers, such as those used in police cars, fire trucks, ambulances, and taxis. These usually have the antenna mounted on the outside of the vehicle.
3. Long-range transmitters and transceivers, such as commercial broadcast transmitters (radio and TV broadcast antenna towers) and amateur (HAM) radios.

**NOTE! Other types of hand-held devices, such as cordless phones, laptop computers, AM/FM radios, TV sets, CD players, and cassette players, and small appliances, such as electric shavers and hair dryers, so far as we know, are not likely to cause problems to the scooter.**

## Scooter Electromagnetic Interference (EMI)

Because EM energy rapidly becomes more intense as one moves closer to the transmitting antenna (source), the EM fields from hand-held radio sources (transceivers) are of special concern. It is possible to unintentionally bring high levels of EM energy very close to the scooter's control system while using these devices. This can affect scooter movement and braking. Therefore, the warnings listed below are recommended to prevent possible interference with the control system of the scooter.



## **WARNINGS**

1. Do not operate hand-held transceivers (transmitters-receivers), such as citizens band (CB) radios, or turn ON personal communication devices, such as cellular phones, while the scooter is turned ON.
2. Be aware of nearby transmitters, such as radio or TV stations, and try to avoid coming close to them.
3. If unintended movement or brake release occurs, turn the scooter OFF as soon as it is safe.
4. Be aware that adding accessories or components, or modifying the scooter may make it more susceptible to EMI.

**NOTE! There is no easy way to evaluate the overall immunity of the scooter.**

5. Report all incidents of unintended movement or braking to your Shoprider® dealer, and note whether there are sources of EMI nearby.

***WARNING: The original manufacturer and the importer of the scooter is not responsible for damage or injury incurred due to improper and unsafe use of our scooter.***



# SHOPRIDER®

## ADJUSTMENTS

### Tiller Angle Adjustment (Fig. 1)

The tiller of your scooter is uniquely designed to allow you to position it to a comfortable driving position. Your scooter's tiller is very versatile, and can be adjusted to different positions. To decide which position best fits your needs, carefully push the black lever that is located on the right side of tiller, and gently lower or raise the tiller. Once you are satisfied with the position of the tiller, simply release the black lever and the tiller will automatically lock for your safety.



Fig. 1

### Seat Slide Adjustment (Fig. 2)

Pulling up on the seat slide lever allows the seat position to be adjusted forward or backward. Release the lever and the seat will lock in that position.

### Seat Recline (Fig. 2)

To manually recline your seat, simply pull up on the lever located on the lower left side of the seat and you will be able to adjust it to fit your desired convenience.



Fig. 2

### Armrest Angle Adjustment (Fig. 2)

Flip up the armrest to allow access to the adjustable roller, rotate the roller to increase or decrease the angle. Your scooter is delivered with the armrests set in parallel to the floor.



# SHOPRIDER®

## SCOOTER TILLER CONTROLS (Fig. 3)


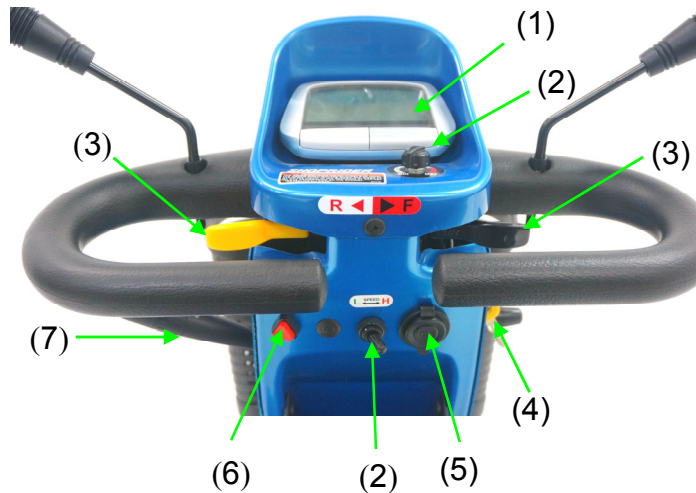


Fig. 3

### (1) LCD Display

Item	Description	Item	Description
1	Speedometer Display (Km/H or MPH)	8	Reset Button (Press & Hold the button to reset trip data to zero under the trip mode)
2	D→Drive , N→Neutral , R→Reverse	9	Mode Button (TRIP/ODO)
3	Odometer / Single ODO Display	10	• Light Flash when low Voltage • Light on: while in charging
4	Electromagnetic Brake Display	11	Headlight (Brightness strong) / Taillight Display
5	Left Turn Signal Display	12	Right Turn Signal Display
6	Headlight (Brightness weak) / Taillight Display	13	Battery Gauge Display
7	On/Off Display (Green On: No Load, Red on: Overload)	14	Light Flashing: Low Voltage Light On: Charger Connected



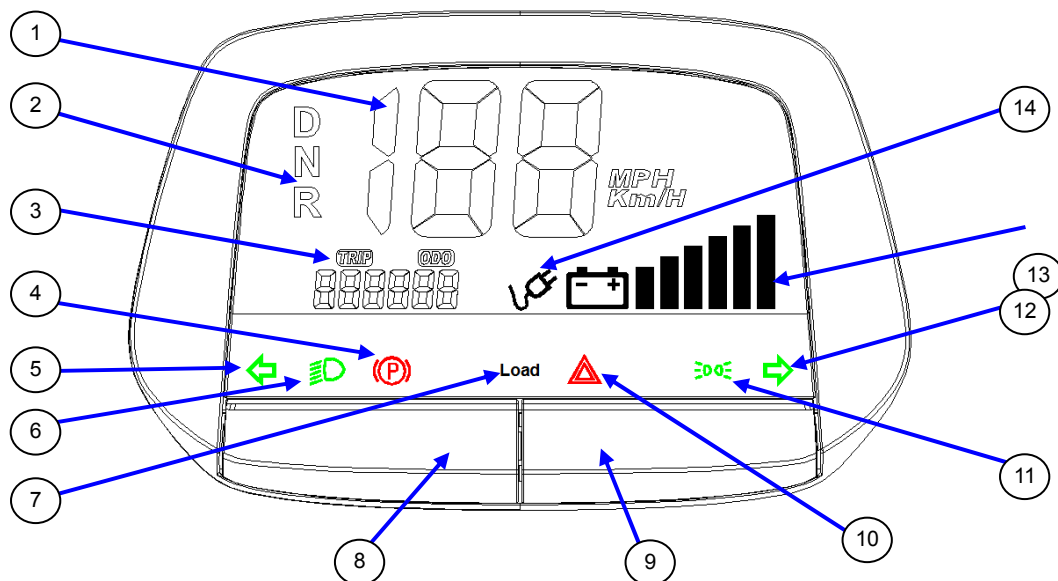

## **(2) Speed Controls**

The speed control allows you to set a speed between 1 (minimum) and 10 (maximum). Turn the dial knob counter-clockwise to “minimum” to decrease speed, and clockwise towards “maximum” to increase your speed. The “High / Low” switch for extra speed. Please use extra caution when using the scooter in “High” mode.

**NOTE:** When attempting to climb slopes, you will need to set the dial to “high” setting. Remember the higher the number you set your speed dial to, the faster your scooter will travel.

## **(3) Thumb/Finger Lever Controls**

The left thumb/finger lever controls the reverse and the right thumb/finger lever controls the forward motion. The amount of pressure applied to each lever will determine the speed the scooter travels at. Release the lever and the scooter will slowly come to a stop and the electromagnetic brake will be applied.

## **Electromagnetic Brake**

The brake is automatically operated when the control levers are released. There is a slight delay when the scooter decelerates; this allows the scooter to come to a smooth stop.

## **(4) On/Off Key Switch**

Insert the key in the right side of the tiller. There is a delay before the thumb/finger lever control can be operated, while the controller is checking the circuits.

## **(5) USB Port**

The USB Port is embedded onto the tiller cover for the convenience of charging 1 set of your USB device, such as mobile phone.


Warning: For your own safety, please be careful if the USB port is used for charging your device while in driving. Do not use your mobile phone whilst driving

## **(6) Hazard Warning Switch**

A red button operates all indicators flashing.

# SHOPRIDER®

## (7) Turn Signal / Light / Horn Lever Switch

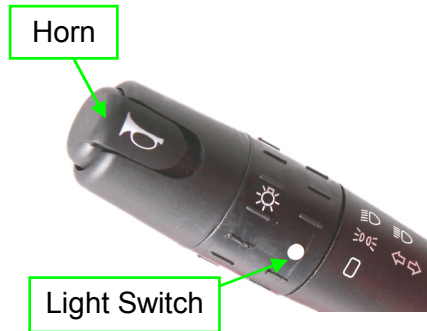

### Turn Signals

The turn signals can be used when the power mode is in ON.

When moved up or down, this lever will cause a light on either the left or right side of your car to flash.

Simply push the signal lever into the neutral position. The neutral between the left and right turn signal positions, and is the default signal ever position on a car.

### Light Switches

Rotating the light switch turns the lights on and off.

1. "☀️" Turns on headlight (Brightness strong) and taillight.
2. "☁️" Turns on headlight (Brightness weak) and taillight.
  - Headlight (Brightness strong): Push the lever forward until you hear a click.
  - Flashing the headlight: Pull the lever back, and release it.
3. "☐" Turns off headlight and taillight

### Horn Button

Pressing the horn button operates an audible buzzer. Use this function to warn pedestrians of your presence when necessary.

# SHOPRIDER®

## FREE WHEEL OPERATION (Fig. 4)

The free wheel release lever is located inside the left rear wheel. To disengage the motor, lift up the lever (the scooter can now be pushed). To re-engage the motor push down on the lever.

***WARNING! Always re-engage the free wheeling device after use. There is No braking on the scooter when this lever is disengaged "Failure to do so may result in injury".***

### Anti-tip wheels

The anti-tip wheels are located at the rear of the scooter frame. The scooter should not be used unless the anti-tipping wheels are in place.

***The original manufacturer and the importer of the scooter is not responsible for damage or injury incurred due to improper or misuse of the scooter.***

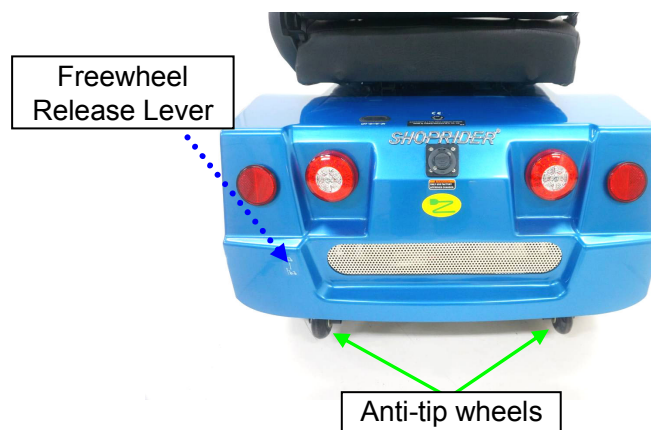


Fig. 4

Risk of injury if the power is switched off while the scooter is in motion, due to it coming to an abrupt, sharp stop.

- If you have to brake in an emergency, simply release the throttle lever and allow the scooter to come to a complete stop.
- If fitted, pull the handbrake until the scooter comes to a stop.
- Only switch the scooter off while in motion as a last resort.



## **STARTING TO DRIVE YOUR SCOOTER**

### **Getting Started**

- Select a level area with plenty of space.
- Mount the scooter, either by lifting the arm and sliding on to the seat or rotate the seat outwards and then rotate the seat back to face the tiller.
- Adjust the tiller and seat as described previously.

### **Starting Scooter Movement**

- Ensure that you are seated in a comfortable position.
- Turn the key to the ON position.
- Your speed setting should be on 1 or 2.
- Pull your right hand finger lever, the scooter will move forward slowly. The harder you pull the faster the scooter goes but will only reach the speed level that has been selected.
- Release the lever and the scooter will slowly come to a stop and the magnetic brake will be applied.

### **Steering**

Steering the scooter is light and easy. Ensure when starting off you give a wide clearance when turning so that the rear wheels clear any obstacle that you are passing. When steering through limited spaces such as a doorway or when turning around, turn the tiller handles where you want to go and apply power gently. The scooter will turn very tightly, but will still move with complete stability as long as the speed is not too high.

### **Reversing**

Before reversing ensure that the area behind is free from any hazards or obstructions. Adjust your speed setting to a low setting before you reverse. Pull/press the left thumb/finger lever, and steer wide of all corners and obstacles.

### **Dismounting**

Before dismounting ensure that the key is turned to the off position.

***CAUTION: Please ensure that care is exercised when entering or leaving the scooter and that the tiller and foot rest is not used for support.***

# SHOPRIDER®

## EXTRA GUIDELINES IN THE CONTROL OF YOUR SCOOTER

### Ramps

The stability of your scooter is governed by several factors such as the seating position, the angle of the slope and your height and weight. When approaching an incline, do so directly and not at an angle and whenever possible avoid making turns. When going down a ramp or slope keep the speed settings to slow. This will ensure there is a safe controlled descent. When the finger levers are released the scooter will slowly stop.

### Safe Climbing Angle

Always follow the guidelines of the safe climbing angle shown as shown in figure 5.

**Caution - Do not turn off the control key while the unit is in motion**

### Grass and Gravel

Please follow the guidelines given previously and your scooter will perform admirably over many surfaces. Avoid long grass, loose gravel and sand.

**NOTE: You should not operate your scooter in wet weather or on wet surfaces. This may cause the motor to sort out.**

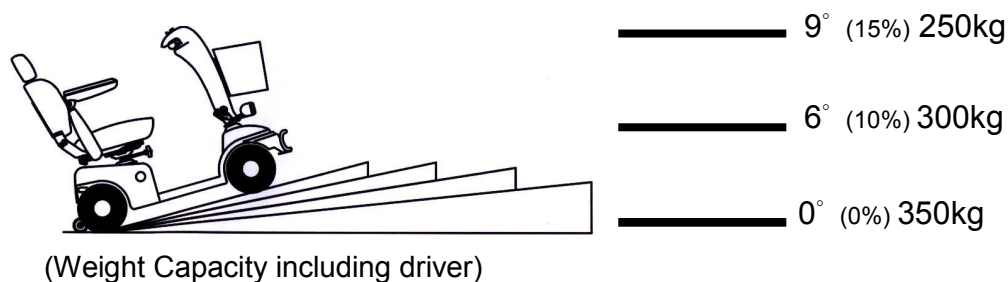


Fig. 5

Angles and weights shown here for this scooter indicate the safe working angles for the user and equipment. Any scooter used outside these parameters may be unsafe and or cause future problems with the electrical systems



## **BATTERIES AND BATTERY CHARGER**

The Shoprider® scooter is supplied with maintenance free, sealed lead acid batteries.

Battery performance can be affected by temperature, terrain, condition of battery, type of battery charger, the weight of the user, wind speed, etc. The battery level indicator is only a guide and is most effective when the scooter is moving, only then is the true discharge level displayed.

- When the scooter stops the battery level indicator may indicate full.
- An active user can use up to 30% of battery capacity or more per day.
- Inactive users may only use 50% of battery capacity in a week.
- In both these instances the battery should be put on charge over night.

### **Battery Charger**

The battery charger supplied is special to your scooter, and may not be suitable for any other power mobility product. Use only the charger supplied with your scooter; other marks of chargers may permanently damage your scooter batteries, and would void the warranty. All the information of battery charger and its operation instruction shall be referred to the user manual attached with the charger in its packaging paper box.

### **First Time Use**

Prior to using your scooter for the first time, you must charge the batteries fully (refer to Battery Charging Procedures). Charging may take up to 16 hours. Charging the batteries completely prior to first time use will benefit battery efficiency. Failure to do so may cause damage.





## Charging

To charge the batteries connect the 3-pin male plug lead into the 3-pin female battery-charging socket located on the rear chassis cover. Plug the A/C cord plug into the wall socket and the other end into the charger (2-pin). Please refer to the LED indicator description based on charging status of the charger's manual. Once the batteries are completely charged, unplug the A/C cord from the wall then unplug the 3-pin lead from the tiller. After use, ensure that the cover is closed back into place. This action helps prevent water from entering the socket connections.

## Off-Board Battery Charger

Your scooter is supplied with an off-board charging facility.

1. Switch your scooter off at the key switch.
2. Open the charge connector cover located on the rear chassis cover and connect charger.
3. Ensure that the charger plug is dry and intact before connecting it to the mains and switching it on.
4. Once the batteries are completely charged, switch off the charger. Unplug the A/C cord from the wall and then press the charger button to unplug the lead from the scooter charging port.

NOTE: Please refer to the LED indicator description based on charging status of the charger's manual.



**WARNING: Only use the original Shoprider® approved charger!**

***This charger is for indoor use only. DO NOT expose it to rain or water spray.***

***Do not leave the charger plugged into the charger port if the charger is not charging.***



## MAINTENANCE

### Servicing

Ensure the scooter is serviced every 12 months to ensure it remains in safe working order failure to do so may cause catastrophic failure with possible injury and damage to surrounding property. Regular servicing will ensure trouble free operation.

### Please Note

Heavy users of the scooter may require more frequent servicing.

Consult with your authorised Shoprider dealer to have the servicing requirements completed every year.

### Lubrication

At six-month intervals you should lubricate the following parts with a light oil or similar lubricant:

- Seat pivot post.
- Seat release lever pinion.
- Wheel bearings.
- Axle of scooter.

**CAUTION – DO NOT USE WD40 TO LUBRICATE PARTS!! DO NOT LUBRICATE TRANSAXLE/GEAR BOX!! In the event of any difficulties please consult your dealer.**

### Cleaning

Your scooter should be cleaned as required however as with all electrical equipment you must ensure that extreme care is used to not water damage any sensitive parts.

Read the following warnings

- **Only** use a soft damp cloth.
- **Never** use a hose to wash your scooter.
- **Do Not** use abrasive cleaning products – Mild detergents only should be used.



## **Wheels and Tires**

The wheels of your scooter have split rims. This allows any punctures to be repaired easily. To change the wheels remove the center 13 mm bolt and slide the wheel off its axles. Take care not to lose the drive key on the rear wheels. When changing a tire you must deflate the tire first. Remove the four bolts to split the rim if required. Ensure that when refitting the wheel the retaining bolt is fitted with a locking washer (with drive key for the rear wheel). Tire pressure is recommended to be kept at 30 psi on the rear tires and 25 psi on the front. Note when refitting the tube a small amount of air in the tube will prevent the tube from pinching when reassembling the split rim.

**Caution: On the tire it states maximum tire pressure is 50psi. This is a maximum tire pressure and the tires should never be inflated to this pressure. 26psi front & 30psi rear should be the maximum the tire are inflated to.**


**Attention: Tires should be maintained at the correct pressure noted above to ensure even wear and ride comfort.**

# SHOPRIDER®

## TROUBLE SHOOTING GUIDE

### No-fuse Breaker

Your scooter is fitted with the latest in electronic controls that are programmed to protect the electrical system from abnormal overloads. Provided for extra protection is a no-fuse breaker toggle switch, located under the seat. Should there be a situation where your scooter stops for no apparent reason, please check that the no-fuse breaker has not tripped.

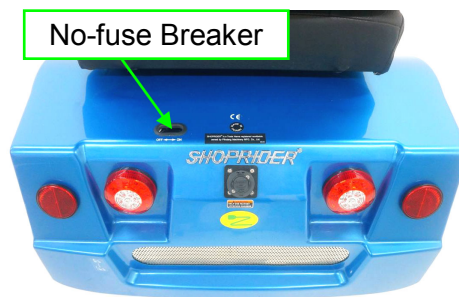

### Check List

If your scooter fails to operate, please check the following:

- The unit is switched on.
- All plugs and connectors are firmly fixed.
- Battery level shows full charge.
- Drive/Freewheel device is in the drive position.
- Circuit breaker has not tripped.
- Battery fuses have not blown.

If the free wheel device has been operated with the scooter left switched on, then the scooter will not operate until the ignition key has been switched off and then on again.

**WARNING! Always re-engage the free wheeling device after use.**

**Failure to do so may result in injury.**

**Please consult your dealer if your scooter does not run after checking these points.**

### Sleep Mode

If the control system is left on and not used for more than 5 minutes, it may automatically “go to sleep.” Simply turn the key off and back on to reset.



## Warranty Terms & Conditions

The Warranty Terms and Conditions noted below provide your **Shoprider®** scooter with excellent coverage against possible defective parts or possible faulty workmanship. If at anytime you have a warranty issue **YOU MUST** contact the **selling Shoprider® dealer** as they are required to undertake the diagnosis and labour content of the warranty. **Shoprider®** will as noted below supply spare parts as required as part of this agreement.

### Structural Frame – 10 Years

Shoprider Australia warrants that the frame will be free from defects under normal use, for a period of ten (10) years from the date of purchase by the original customer on units purchased after October 2022. Units sold prior to this date attract a three (3) year warranty on this part. Shoprider Australia will repair or replace, at our option free of charge any part found on examination by Shoprider Australia to be defective in material and or workmanship within the warranty period.

Ferrous Corrosion to the frame or Damage from use of the scooter where damage to the frame is caused by non-standard or incorrect storage or use is not covered under the warranty.

All parts claimed under warrant must be returned to Shoprider Australia in their present condition. Only an authorised “Warranty Dealer” of Shoprider Australia can undertake warranty work. Always contact the selling Shoprider® dealer to arrange all work and assess if warranty applies.

### Drive Train Assembly – 18 months

Shoprider Australia warrants that the drive train Assembly will be free from defects under normal use, for a period of eighteen (18) months from the date of purchase by the original customer.

Shoprider Australia will repair or replace, at our option free of charge any part found on examination by Shoprider Australia to be defective in material and or workmanship within the warranty period.

All parts claimed under warrant must be returned to Shoprider Australia in their present condition. Only an authorised “Warranty Dealer” of Shoprider Australia can undertake warranty work.

Always contact the **selling Shoprider® dealer** to arrange all work and assess if warranty applies.

### Drive Train Components include

Transaxle, Motor and Magnetic Brake only



## Warranty Terms & Conditions continued

### Other Parts – 12 months

Shoprider Australia warrants that the other parts will be free from defects under normal use, for a period of twelve (12) months from the date of purchase by the original customer.

All parts claimed under warrant must be returned to Shoprider Australia in their present condition. Only an authorised “Warranty Dealer” of Shoprider Australia can undertake warranty work.

Always contact the **selling Shoprider® dealer** to arrange all work and assess if warranty applies.

### Exclusions

1. All transportation cost associated with the submission and return of the repair or replacement under warranty (as determined by Shoprider Australia) are the responsibility of the original customer.
2. Normal wear items are not covered under warranty. These items include such items as brushes, tires, tubes, brake pads, upholstery, seating and armrests.
3. Damage caused by
  - Excessive, abusive, misuse, negligence, accident or water ingress or corrosion
  - Improper maintenance, maintenance carried out by a non approved “Warranty Dealer”
  - Improper operation & storage
  - Any **commercial** use what so ever
4. Where an authorised dealer has not lodged all required documentation at the appropriate time, the warranty is Void
5. All warranties provided do not extend to purchasers of used or secondhand equipment  
**(Warranty's are not transferable)**

### Please Note

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonable foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to a major failure

**NOTE:** In the event of a warranty issue, Always contact your **selling Shoprider® dealer** who will arrange any work and return any faulty parts. Parts will be assessed and replaced within 3-5 working days.



## SPECIFICATIONS

Model Name		ROCKY 12
Overall Dimensions: L <sup>1</sup> x W x H	mm	1740 x 750 x 1310
Front Tire Diameter	pcs x in	2 x 16"
Rear Tire Diameter	Pcs x in	2 x 16"
Weight Capacity including driver (on the Level Road)	kg	350
Total Weight (w/ Batteries, Basket, Seat)	kg	186
Battery Capacity	_V_Ah x Pcs	12V 100Ah x 2
Charger	Amp	8A
Output Power of Motor (Reference Only)	W(Hp)	1500(2)
Max. Speed <sup>2</sup>	kph	10~13
Safe Climbing Angle	Deg. (%)	Fig.5
Range <sup>3</sup> (Per charge w/ standard battery)	km	50
Turning Radius	mm	2705

**\*\*\*ALL SPECIFICATIONS ARE SUBJECT TO CHANGE WITHOUT PRIOR NOTICE\*\*\***

1. To include the anti-tip wheels
2. Driver weight may exceed weight of the unit, the speed must be reduced when turning.
3. The actual driving range varies with the factors shown below:
  - a) the weight of occupant
  - b) battery conditions
  - c) type of charger
  - d) ambient temperature
  - e) the way of driving
  - f) terrain
  - g) the wind speed
  - h) etc.



### Manufacturer:

PIHSIANG MACHINERY MFG. CO., LTD.  
 No. 108, Hsin-He Road,  
 Hsin-Feng Hsiang  
 304 Hsinchu County, Taiwan





## NOTES



## DISCLAIMER

Congratulations on your purchase of the SHOPRIDER® Scooter.

This Scooter is not intended to be used by individuals with physical limitations that could prevent the user from operating the Scooter safely. SHOPRIDER® disclaims all responsibility for any personal injury or property damage, which may occur as a result of improper or unsafe use of its products.

The preceding guidance is intended to assist you in the safe operation of this Scooter.

If you should have any questions about the correct operation of the Scooter, please contact your authorised SHOPRIDER® provider.

Please complete the information below for future reference

Shoprider Selling Dealer -	_____
Scooter Model Number -	_____
Scooter Serial Number -	_____
Date of Purchase -	_____

**ATTENTION:** This owner's manual is printed based on the latest specifications and product information available at the time of publication. SHOPRIDER® reserves the right to make changes as they become necessary. Any changes to the SHOPRIDER® range may cause slight variations between the illustrations and explanations in this manual and the scooter model that was purchased. Where possible a newer version of this manual may be available on our website [www.shoprider.com.au](http://www.shoprider.com.au)