

**REDGUM** BRAND  
Daily Living and  
Lifestyle Products

---

**Serena Wheelchair**  
**Tilt In Space**  
RG15-18TIS / RG19-21TIS

**Crash Tested Chair**  
**(ISO 7176-19)**  
**See Page 3 for detail**



---

Phone: 08 9248 4180  
Fax: 08 9248 9452  
E-mail: [orders@redgumbrand.com.au](mailto:orders@redgumbrand.com.au)

**REDGUM** BRAND  
Daily Living and  
Lifestyle Products

*REDGUM Brand is a 100% Australian Owned & Operated*

## Serena Wheelchair

### Tilt In Space

RG15-18TIS / RG19-21TIS



### Features

- Seat & Backrest adjustable to any angle
- Detachable Armrests
- Armrests adjustable for height & depth
- Elevating leg rests are
  - ⇒ Detachable
  - ⇒ Horizontally adjustable
  - ⇒ Calf support is fully adjustable
- Standard seat height of 530mm
- Height adjustable handle bar
- Locking brake with drum hand brake
- 24" wheels with solid tyres
- Quick release rear wheels
- Anti tip wheels
- 8" Front rubber tyres
- Dartex covered cushions
- Fully adjustable head support
- Adjustable side support pads
- Abduction wedge
- Adjustable width frame
  - 380-450 or 480- 530mm

### Dimensions

- Seat width
  - ⇒ 380-450 or 480- 530mm
- Seat depth 410 to 505mm
- Seat height 530mm
- Seat angle -8 to 30'
- Back rest height 530 to 580mm
- Back rest angle -9 to 30'
- Armrest height 175 to 280mm
- Overall width 610 to 690mm
- Overall height 1060mm
- Overall Length 1310mm
  - ⇒ with no leg rests 930mm
- Max user weight 130kg
- Weight of chair 35.5 to 38.5kg

### Crash Rating

Crash Tested Chair (ISO 7176-19)

See page 3 for detail



## Serena Wheelchair Tilt In Space

RG15-18TIS / RG19-21TIS

### Transporting with a passenger

This wheelchair meets the standard for being used as a seat within a authorised transport vehicle (ISO 7176-19)

**Please note** - the vehicle being used to transport the wheelchair must be fitted with the correct rated safety straps and vehicle restraints for that vehicle. (this includes at a minimum a rated lap sash belt restraint)

It is also important to be aware of the nominated tie down points on the Serena Tilt In Space Chair -

Refer the images below for the locations of these points



**Front Castor Tie Down**



**Rear Bar Tie Down Point**



**Backrest Tie Down Point**